


Administering Intramuscular (IM) and Subcutaneous (SQ) Injections

01/ Attach Luer Lock Adaptor to injection needle




02/ Remove cap from Syringe Unit

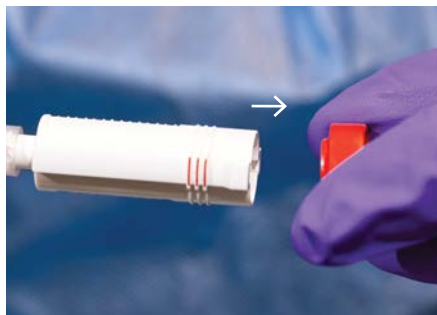

03/ Disinfect Luer Lock Adaptor membrane per facility protocol



04/ Align red to red and engage syringe unit to Luer Lock Adaptor (Push On)

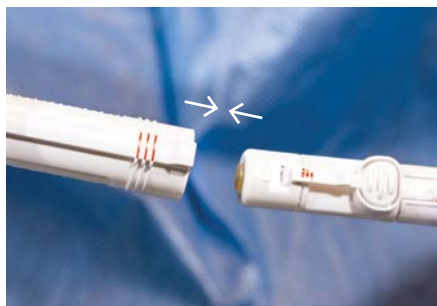

05/ After injection is complete dispose of components per facility protocol



***NOTE:** Facility protocol may vary. Please follow local safe handling guidelines and your facility protocols for aseptic handling.

Administration

EQUASHIELD®
Compounding Technologies

2 Harbor Park Drive, Port Washington, NY 11050 | www.equashield.com
Clinical Support Line: 516-684-8200 | **Fax:** 516-684-8202

EQL-1603.7 2023-01 | © 2023 Equashield, LLC