

Administration Components



LL-2

Luer Lock Adaptor



LL-2S

Luer Lock Adaptor-Swivel



FC-1

Female Connector



FC-1S

Female Connector-Swivel




*Luer LL-2 / LL-2S to access site or injection needle; disinfect membrane per facility protocol



*Align red to red and engage Female Connector to Luer Lock Adaptor




Administration Components

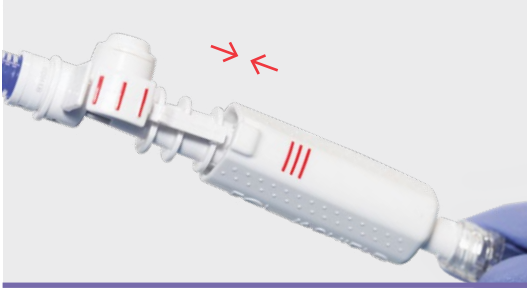

Engage

All Equashield components are engaged by aligning the red marks on each component and sliding components together (Push On)

DO NOT TWIST OR ROTATE SYSTEM COMPONENTS DURING OR AFTER ENGAGEMENT

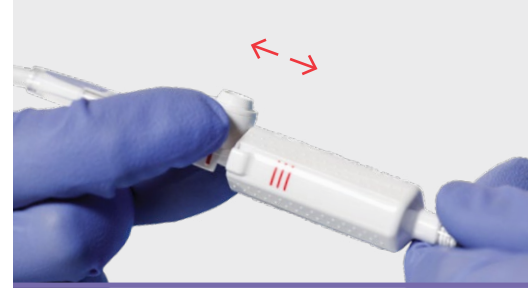

Disengage

All Equashield nursing components are disengaged by depressing the lever on the Luer Lock Adaptor (LL-2/LL-2S) and pulling apart (Pull Off)